

ProLoquo2Go (v3.0.3) and Pictello (v2.0) on iDevice (iOS 7.0.4)

Using ProLoquo2Go to Write into Pictello

Introduction

Proloquo2Go is a communication app for the Apple iPad (iPad mini, iPod touch and iPhone). Communication messages or text created in the Proloquo2Go app can be exported into Pictello to create a story, make journal entries or create a personalized learning log for science or socials.

This tutorial assumes the reader should already know how to:

- create messages in ProLoquo2Go,
- take photos on an iDevice or save images to the camera roll,
- read a story in Pictello.

Learning Objectives




Completion of this tutorial will give you experience with the following:

- Creating a new story in Pictello using the Wizard
- Copying messages created in ProLoquo2Go into Pictello
- Navigating between apps

Case Study

Samantha is a 10 year old student with Down Syndrome. She is non-verbal. She has an interest in books but cannot yet read on her own. She likes to draw pictures and write although her output is not recognizable. Samantha is beginning to use ProLoquo2Go. She can label objects and is beginning to put 2-4 words together in telegraphic utterances during structured practice (e.g.: I see _____. I like blue ball. I go on swing.). She has the Pictello app and has been using it to read Accessible Books. Her team would like to expand her use of these two apps so she can write her own books.

Create a title for your Pictello story in ProLoquo2Go

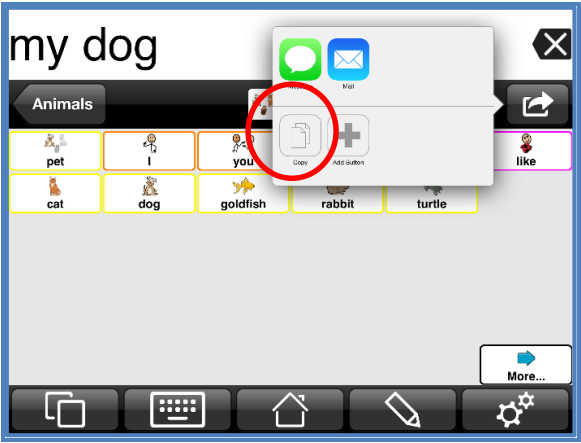1	Open Proloquo2Go.	
2	Write the name of your story. Tap the right arrow in the top right corner of the taskbar.	
3	A pop-up screen will appear. Select Copy.	

4 **Minimize** Proloquo2Go by **double pressing** the iDevice **Home** button.






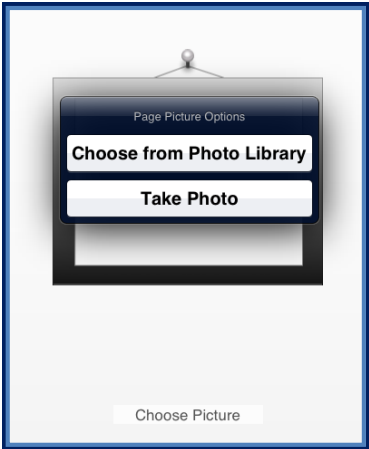
Create a story in Pictello

1 **Open** the Pictello app.




2 **Select the “+” sign** in the bottom taskbar to add a new Pictello story then **select the Wizard**.

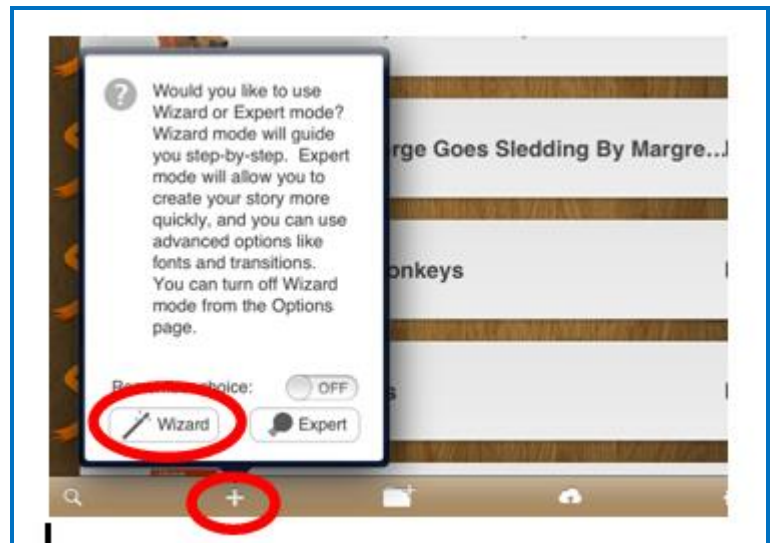


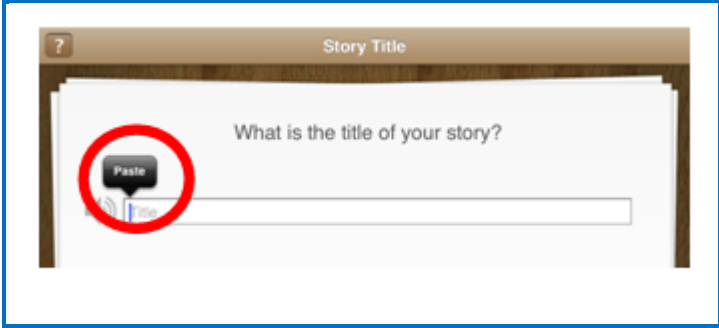
<p>3</p>	<p>Hold your finger on the textbox for the title of your story and then release, select Paste.</p>	
<p>4</p>	<p>Close the keyboard pop-up. Then select the right arrow in the bottom taskbar to navigate to the next step.</p>	
<p>5</p>	<p>Tap the centre of the picture frame and a pop-up menu allows you to select “Choose from Photo Library or Take a Picture”.</p>	

6 Select your **picture** if using your Camera Roll.

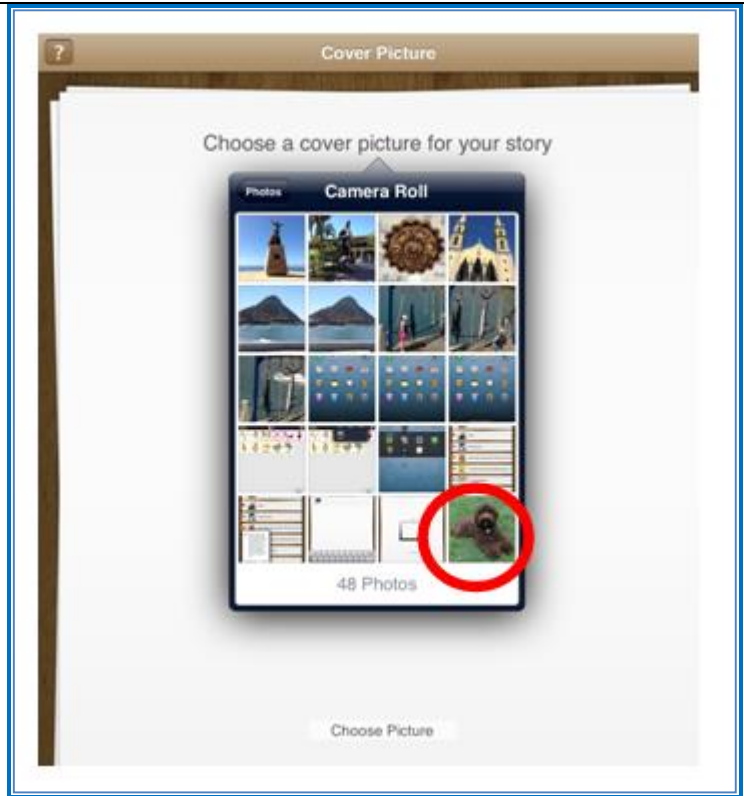

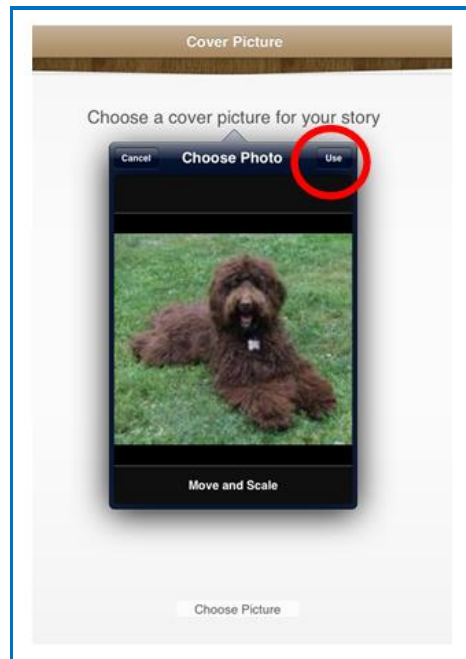

7 In the Choose Photo pop-up, **select Use**.

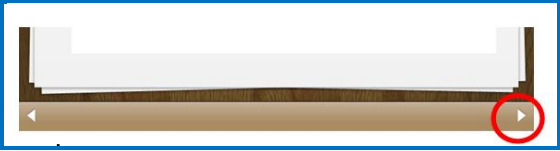


<p>8</p>	<p>Once the picture shows up on the title page, navigate to the next step by selecting the arrow in the bottom right hand corner of the taskbar.</p>	
-----------------	--	--




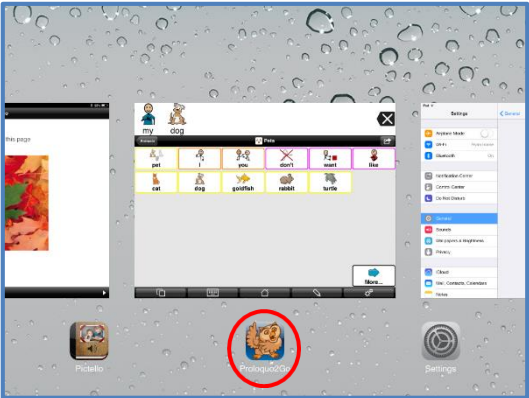
Add sound

<p>1</p>	<p>Choose the voice you want for your story or record what you have written.</p>	
-----------------	--	---

<p>2</p>	<p>Select the right arrow in the bottom taskbar to move to the next step.</p>	
-----------------	--	--


Add more pages to your story
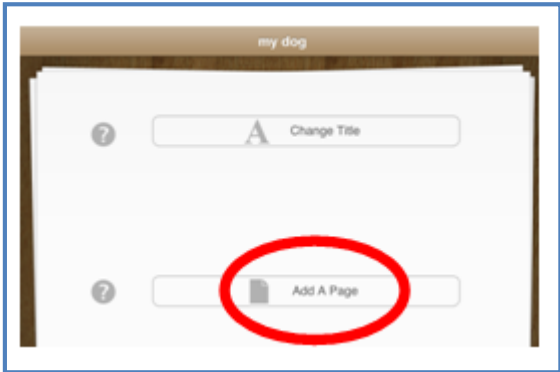
<p>1</p>	<p>Select "Add a Page".</p>	
-----------------	------------------------------------	--
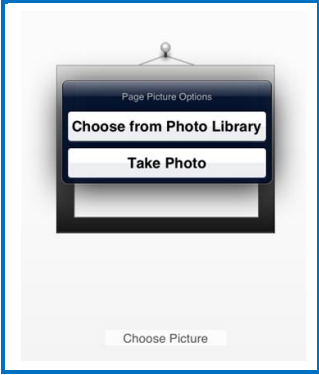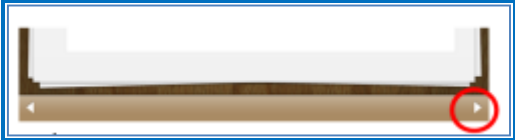
<p>2</p>	<p>Add the next photo in your story as you did for the title cover photo. Select your photo source and then the specific photo.</p>	
<p>3</p>	<p>Select the right arrow in the bottom taskbar to move to the next step. Skip the voice page unless you want to change the voice or record.</p>	
<p>4</p>	<p>Think of the text you want to match the page.</p>	
<p>Back to ProLoquo2Go to formulate text</p>		
<p>1</p>	<p>Double press the iDevice Home button and select ProLoquo2Go from the open apps.</p>	


2

Clear the title from the **message window** by selecting the “X” in the right corner.

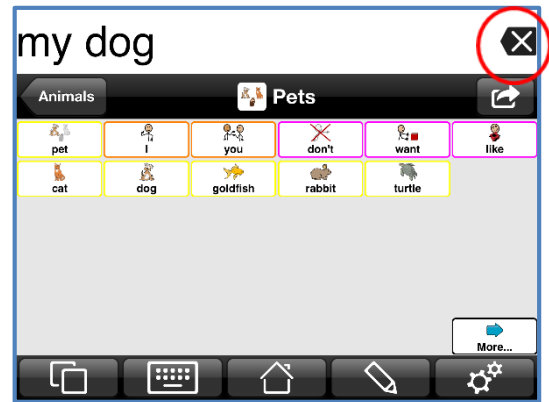


3

Formulate your message (one or several lines of text). **Tap the right arrow** in the top right corner of the taskbar. A pop-up screen will appear. **Select Copy**.

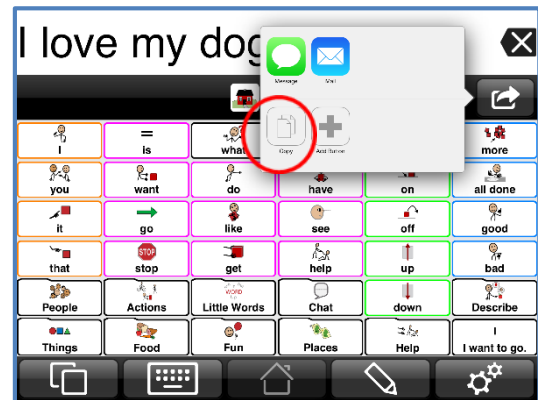

4


Minimize Proloquo2Go by double pressing the iDevice Home button. **Select Pictello** from the open apps.

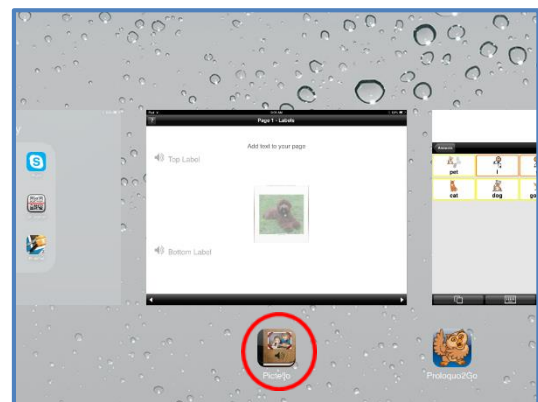

Back to Pictello


1

Select either the top or bottom **label** for your text. **Hold your finger** on the textbox and then **release, select “Paste”**. Your text will print from ProLoquo2Go. **Close the keyboard** pop-up and select the right arrow in the bottom taskbar.

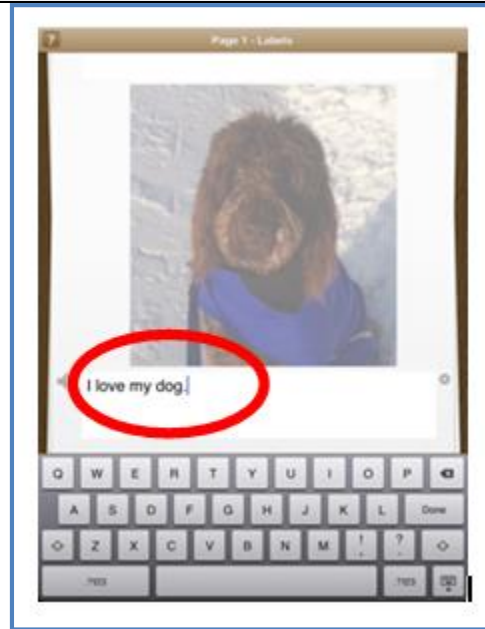

3


Continue adding pictures, navigating to ProLoquo2Go to write, copying and pasting the text. **When finished** with your story, instead of adding another page, **select “Done With Story”**.

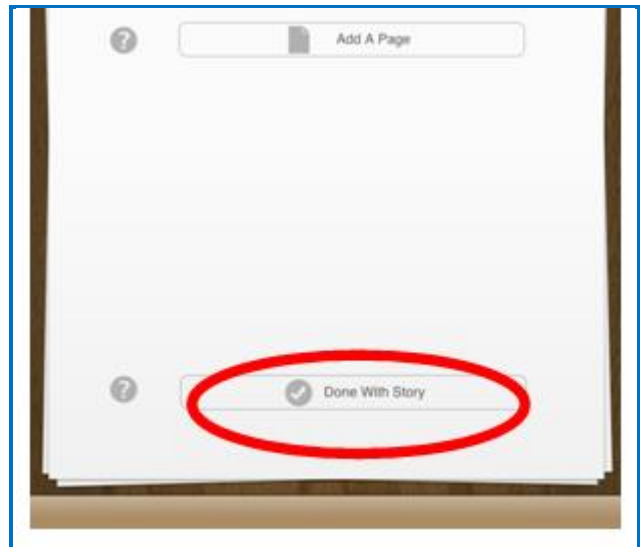

Read, Edit, Share, Export or Print your Story


1

Your story will now be listed as the last item in the Pictello Library. You can **read** it by **pressing** the **arrow** to the right of the title.

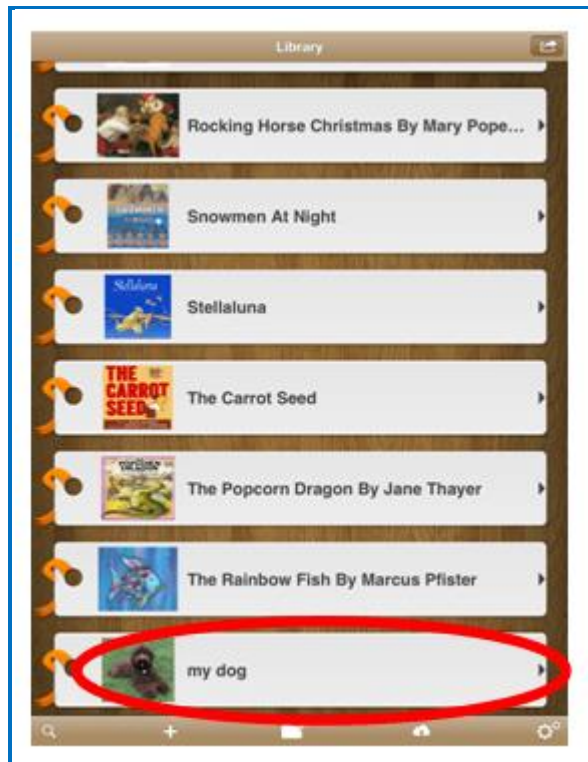


2

If you want to **edit, share, export as a pdf,** select the **button** in the **top right corner** of the Pictello taskbar.

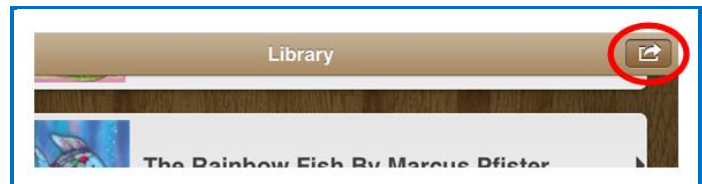

3

Select your choice and when finished, select Done from the top right corner to have access to the reading function again.

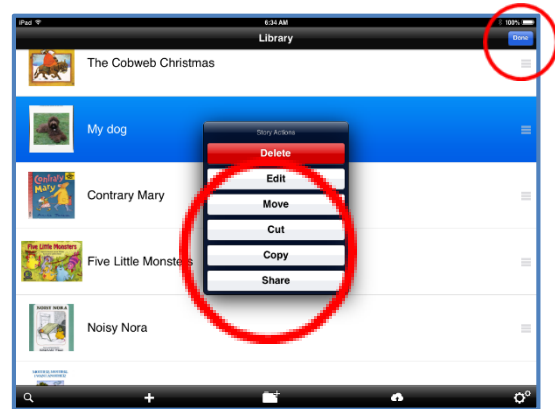

Tip...Some teams have found it helpful to create the photo content for stories in Pictello first and then add the text later by navigating between the story and Pictello. One caution in this approach is that you will need to create your title for your story before adding the pictures. This can be done by typing if desired.