

## MagniLink Keyboard Commands

Function	Mac	PC
<b>Natural Colours</b>	fn + F1	F1
<b>Positive Picture with Increased Contrast</b>	fn + F2	F2
<b>Negative Picture with Increased Contrast</b>	fn + F3	F3
<b>Take a Snapshot</b>	fn + F5	F5
<b>Take a Snapshot – Fast</b> (saves to last folder or MagniLink folder)	fn + Shift + F5	Shft + F5
<b>Start/Stop Video Recording</b>	fn + Alt + F5	Ctrl + F5
<b>Start Video Recording – Fast</b> (saves to last folder or MagniLink folder)	fn + Alt + Shift + F5	Ctrl + Shift + F5
<b>View Stored Snapshots</b>	fn + F6	F6
<b>View Last Stored Snapshot – Fast</b>	fn + Shift + F6	Shift + F6
<b>View Stored Videos</b>	fn + Alt + F6	Ctrl + F6
<b>View Last Stored Video – Fast</b>	fn + Alt + Shift + F6	Shift + Ctrl + F6
<b>Preset Split Modes</b>	fn + F8	F8

<b>Information and Setting Window</b>	fn + Alt + F9	F9
<b>Return to Default Settings</b>	fn + Alt + F11	F11
<b>Zoom In</b>	Arrow Up	Arrow Up
<b>Zoom Out</b>	Arrow Down	Arrow Down
<b>Overview</b>	Alt + O	Ctrl + O
<b>Decrease Brightness</b>	Arrow Left	Arrow Left
<b>Increase Brightness</b>	Arrow Right	Arrow Right
<b>Guiding Line Left/Up</b>	Alt + Arrow Left	Ctrl + Arrow Left
<b>Guiding Line Left/Up</b>	Alt + Arrow Up	Ctrl + Arrow Up
<b>Guiding Line Right/Down</b>	Alt + Arrow Right	Ctrl + Arrow Right
<b>Guiding Line Right/Down</b>	Alt + Arrow Down	Ctrl + Arrow Down
<b>Toggle “always on top”</b>	Alt + A	Ctrl + A
<b>Minimize MagniLink</b>	Cmd + M	Ctrl + M
<b>Rotate the Camera Picture Clockwise</b>	Alt + R	Ctrl + R
<b>Rotate the Camera Picture Counter Clockwise</b>	Shift + Alt + R	Shift + Ctrl + R

<b>Mirror Camera Image</b>	Alt + I	Ctrl + I
<b>Freeze Camera Image</b>	Alt + F	F
<b>Autofocus On/Off</b>	Alt + J	Ctrl + J
<b>Open the Preferences Dialog</b>	Cmd + ,	
<b>Open the Manual</b>	Cmd + ?	
<b>Global Short Cut Commands</b>		Ctrl + Alt + S
<b>OCR-Process</b>	fn + F4	F4
<b>Process with OCR and Add Text</b>	fn + Alt + F4	Ctrl + F4
<b>Toggle Column Selector</b>	Alt + S	Ctrl + S
<b>Copy Text</b>	Cmd + C	Ctrl + C
<b>Paste Text</b>	Cmd + V	Ctrl + V
<b>Paste and Add Text</b>	Cmd + Alt + V	Ctrl + Shift + V
<b>Open Text Document</b>		Alt + F6
<b>Switch Between Camera View and Text View</b>	V	V
<b>Display Mode</b>	B	B
<b>Start/Pause Reading</b>	1	1
<b>Stop Reading</b>	2	2
<b>Read Previous</b>	3	3
<b>Read Next</b>	4	4

<b>Switch Reading Mode</b>	5	5
<b>Slower</b>	6	6
<b>Faster</b>	7	7
<b>Reduce Volume</b>	8	8
<b>Increase Volume</b>	9	9
<b>Next Voice</b>	0	0