

Adding Camp pages to your iPad

Download the Camp.prt file to your computer.

Open iTunes on your computer. Connect it to your iPad with the USB cord.

Select the Apps drop down menu.

Scroll down and select TouchChat Suite.

Select Add from the bottom of the TouchChat Documents window and browse to find the Camp.prt file and select it. It will upload to this window.

Next go to your iPad and open TouchChat Suite and open your user set up.

Select the Choose Vocab button on the left on your home page.

Select the Menu button and Import Vocabulary.

Select the Camp.prt file... it will become available in your Vocabularies list.

Then go back into your user's setup home page.

Select Menu→Edit Page→Import/Copy pages→Import another page

Select the Camp file and select all the pages marked "camp" (not Home)

Then select Import→Done

Now link these pages into your user's setup using the "Camp Options" page will allows you access to all the pages.